
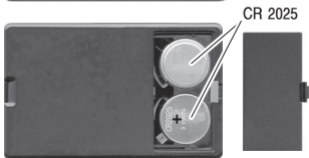
easyPOCKET

- Bedienungsanleitung
- Operating instructions
- Mode d'emploi
- Istruzioni d'uso
- Instrucciones de uso
- Gebruiksaanwijzing
- Brugsanvisning
- Bruksanvisning
- Käyttöohje
- Návod k použití
- Instrukcja obsługi
- サービスマニュアル

 **ESCHENBACH**

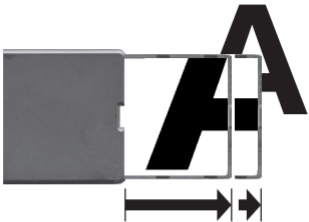CE

Deutsch

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English

Safety instructions

- Danger of fire! Lenses used in optical

equipment may cause considerable damage through their 'burning glass effect' if used or stored incorrectly. Ensure that optical lenses are never left uncovered in the sunlight!

- Danger of blinding or injury Never look at the sun through any optical equipment! Do not look directly at the light sources! Risk group 1 as per EN 62471:2008.
- Protect your magnifier from knocks, bangs, moisture and excessive heat. Never place your magnifier on heaters or in the sunlight.
- Ensure that other people, especially children, are aware of these dangers!

Care Instructions

- ▶ When cleaning the easyPOCKET do not use any soap solutions that contain softening agents, no alcohol-based or organic solvents and no abrasive detergents! The lenses could be destroyed!
- ▶ Do not clean the easyPOCKET in an ultrasonic bath or under flowing water!

- ▶ Clean the easyPOCKET with a soft, lint-free cloth (e.g. with a cleaning cloth for glasses). For heavier soiling (e.g. fingerprints), moisten the cloth slightly.

Disposal

When the device reaches the end of its useful life, do not dispose of it with the normal household waste.

Batteries and cells must not be disposed of in household waste! Ensure that they are disposed of properly!

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Warranty

Within the framework of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing errors or material errors. In event of damage due to improper handling, including damage from a fall or impact, no warranty claim exists. Warranty claims are only accepted upon presentation of the sale receipt!

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Eschenbach Optik GmbH